





NEW FUNCTION INSTRUCTIONS

Working Mode Instruction:

MODE	DESCRIPTION	MARK
FULL POWER	constant light power output	
PULSE	pulse mode light output	
PROGRESIVE	light output keeps gradually increasing and gets constant to full power after 5s	
CYCLE	User sets a working time, when it is over, it stops a few seconds, then repeat the previous setting working time, and keeps repeatedly working like this. Different working time and different power level, the time intervals are different, see the table below.	

Time Interval in different power levels and working times:

Working Time Power Level	1S	2S	3S	4S	5S	10S	15S	20S	25S	30S	35S	40S
P1	3	3	4	/	/	/	/	/	/	/	/	/
P2	3	3	3	3	3	4	5	6	/	/	/	/
P3	3	3	3	3	3	4	4	5	5	6	6	7

Mode Switching Operation

Press the button "Δ" about 1 second, release after one beep, the working mode was switched to the next one. The order is: FULL POWER – PULSE – PROGRESIVE –CYCLE – FULL POWER – ...

Power Level Switching Operation

Press the button "Δ" about 2 seconds, release after two beeps, the power level was switched to the next one. The order is: P3 – P2 – P1 – P3 – ...